



Instructions for use:

1. Hygiene of eyelids and eyelashes: Open a sachet and unfold the wipe inside. Keeping the eye closed, gently apply the pad to the eyelids and the base of the eyelashes. Carefully massage the eyelids with small circular movements, gently removing any crusts, secretions or make-up.

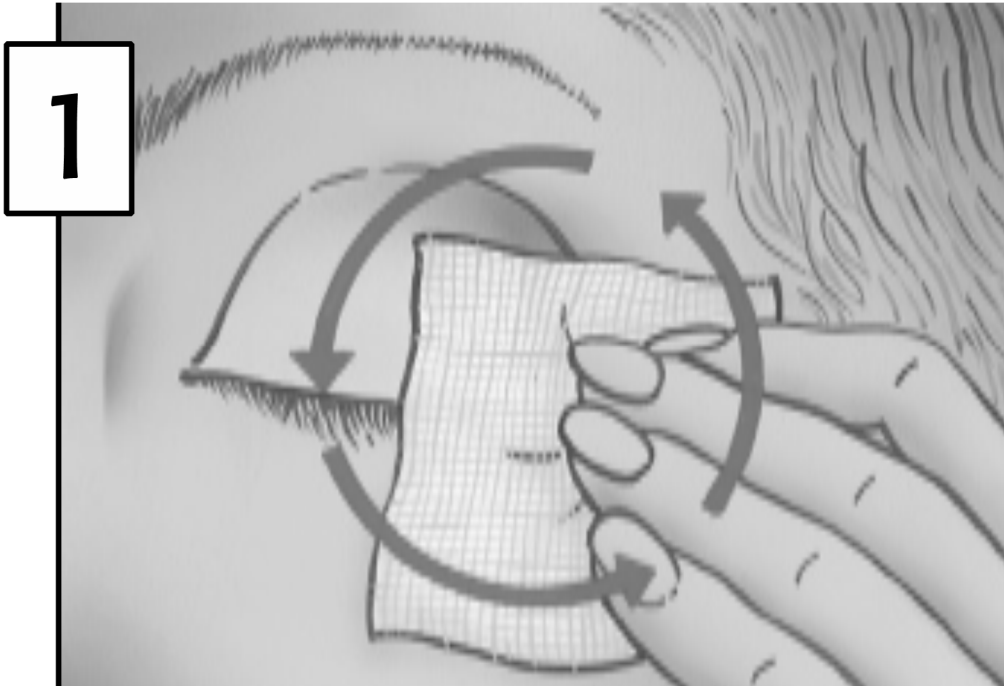

2. Hygiene of the eyelid margin: In front of a mirror, carefully open the eyelids by lightly pulling on the lashes. Wrap the wipe round your first (index) finger and carefully rub the eyelid margin. Repeat on the other eye, using a new wipe.

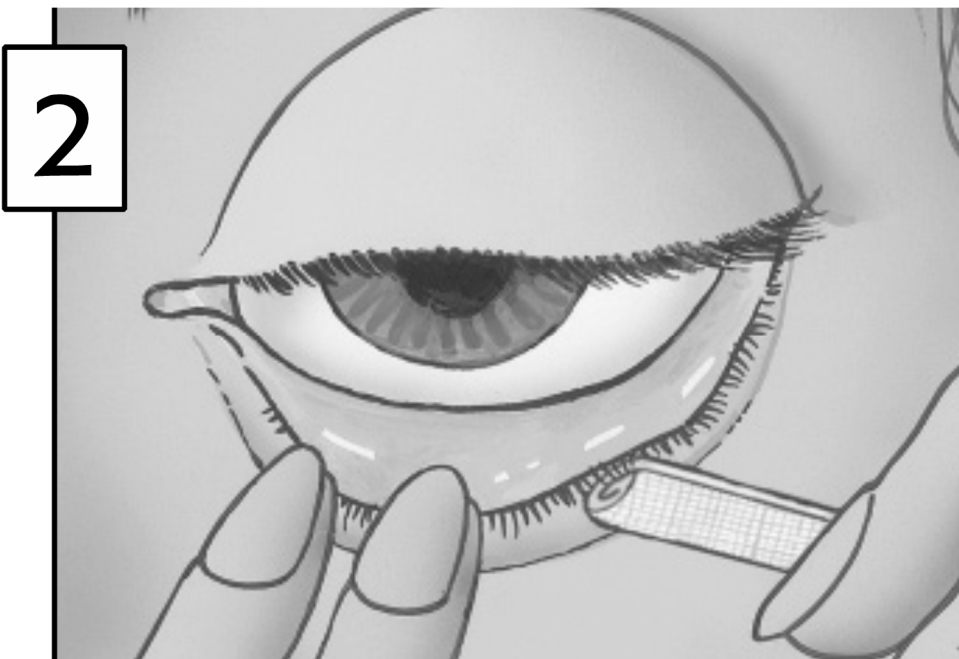

Precautions for use and warnings:

For use on the skin only, do not put directly into the eyes.

Do not use if you are allergic to any ingredient.

BLEPHACLEAN is a sterile product, **for single-use only**.

Do not use again, as this may mean the pad works less well and can increase the risk of infection.

Do not use the wipe if the sachet is opened or damaged.

The colour of the wipe (lotion) may vary. This is due to natural plant extracts and is normal.

Stop using Blephaclean and contact your doctor, optometrist or pharmacist if your symptoms get worse.

Do not use after the expiry date clearly marked on the outer box and on the sachet. The expiry date refers to the product in intact packaging, stored correctly.

Any wipe should be thrown away after use into the household waste.

Store between 8 and 25°C.

KEEP OUT OF THE SIGHT AND REACH OF CHILDREN.

Possible side effects:

The following events have been reported with Blephaclean: burning or itching sensation for some minutes after use, redness or swelling of the eyelids, allergic reactions.

Product description:

Sterile wipe, impregnated with a lotion.

Lotion contains: Capryloyl glycine, Sodium hyaluronate, PEG 8, Poloxamer 184, Polysorbate 20, Propylene glycol, hydrocotyl extract (propylene glycol, *Centella asiatica* extract and water), Preservative-free Sebum-regulating complex (propylene glycol, *Iris florentina* root extract, zinc sulphate, PPG-5 Ceteth 20 and Retinyl palmitate (Vitamin A), alcohol and water), PEG 6 caprylic/capric glycerides (i.e. Macrogol 6 glycerol caprylocaprate), Dipotassium phosphate, Sodium hydroxide, Potassium dihydrogen phosphate, Purified water.

Date of revision of the instruction for use: March
2016

Marketed in the UK and Ireland by:

THEA Pharmaceuticals Ltd

IC5 Innovation Way, Keele University Science and
Business Park, Keele, Staffordshire ST5 5NT UK

www.thea-pharmaceuticals.co.uk

From UK : 0870 192 3283

From outside : UK +44 (0) 870 192 3283




Manufacturer

Laboratoires THEA

12, rue Louis Blériot

63017 Clermont-Ferrand Cedex 2

France

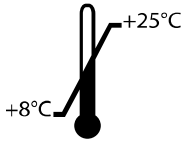

Store between 8 and 25°C




Do not re-use



Do not use if package (sachet) is damaged



Sterilised using irradiation

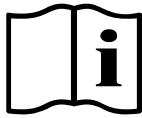

Consult instructions for use



Caution



Use-by date



Batch number

